

Bed-wetter/waterproof Mattress

Vynmed™ product cleaning instructions:

PLEASE READ INFORMATION AND ALL CLEANING INSTRUCTIONS PRIOR TO USE OF YOUR NEW MATTRESS

*North America Mattress Corp. recommends you keep these instructions for future reference.

**If using a different cleaning solution than recommended, first spot test an inconspicuous area before cleaning effected surfaces. If in doubt please contact us at 1-800-448-6163 and a service specialist will gladly assist you. We are open Monday-Friday from 8AM-4:30PM PST

Product care recommendations and process summary:

- **CLEAN** with mild liquid soap concentrate (1-2 drops soap to 1 quart water), then **RINSE** with clean water
WARNING – please DO NOT USE VINEGAR SOLUTIONS – vinegar will eat away at the material due to its high acidity
- **DISINFECT** with gentle disinfectant (spray bottle) then **RINSE** with clean water

REMEMBER - THE KEY IS TO REMOVE AND CLEAN ALL FLUIDS, THEN DISINFECT AND RINSE

CLEANING STEPS

STEP 1 – REMOVE FLUIDS

First, remove any sheets or covers to reveal soiled surfaces (if soiled area has dried, then pre-wet with clean lukewarm water and gently scrub with soft-bristled brush or non-abrasive scrubbing pad). **Second**, soak up liquids with non-abrasive cloth or paper towels and wipe clear all fluids from effected areas.

NOTE: Be sure to check all surfaces (top, sides and bottom) for fluids to ensure an effective cleaning and disinfecting result

STEP 2 – PREPARE CLEANING SOLUTION AND CONTAINERS

Create mixture of mild soap and lukewarm water in one bucket or container (mixture of 1 to 2 drops soap concentrate to 1 quart lukewarm water) and fill a second bucket or container with clean water.

NOTE: North America Mattress Corp. recommends use of mild liquid soap for safe and effective results

STEP 3 – CLEAN SURFACE

Wipe surfaces down thoroughly with non-abrasive cloth soaked in cleaning solution.

STEP 4 – RINSE & WIPE DRY

First, soak a new non-abrasive cloth in clean water and wipe down surfaces to remove residual cleaning solution.

Second, use dry clean non-abrasive cloth to wipe away residual water and any remaining cleaning solution.

DISINFECTING STEPS

STEP 1 – APPLY DISINFECTANT

First, with soiled surfaces properly cleaned, use a trigger-action spray pump to apply disinfectant to all previously cleaned areas. **Second**, wipe disinfected across all treated surfaces with non-abrasive cloth to ensure any bacteria spread during cleaning is neutralized—you can never be too careful.

NOTE: North America Mattress Corp. recommends use of gentle disinfectants for safe and effective results

STEP 2 – RINSE & WIPE DRY

First, soak a fresh non-abrasive cloth in clean water and wipe all treated areas clear of disinfectant. **Second**, wipe dry with new non-abrasive cloth. Let dry before replacing bed sheets.

